- In the Inlay. Place inlay between shoeleaves and shoe tongue, with shortest end of inlay towards toe-end (Fig. 7b).
- Thread the Connexion Strap (F) through the heel part, central, or lateral configuration (Fig. 7c).
- Snap Keyhole (E) into place over nut (Fig. 8a, Fig. 8b, Fig. 8c).
- Thread Connection Strap (F) through Ankle Cuff Buckle (C) and pull to tighten (Fig. 9a, Fig. 9b, Fig. 9c).
- The applied version of the device should look as shown in the figures (Fig. 10a, Fig. 10b, Fig. 10c).

### Device Removal

Snap keyhole off the nut. Adjustment of the Connexion Strap should not be necessary for subsequent application.

### Accessories and Replacement Parts

Please refer to the Ossur catalog for a list of available replacement parts or accessories.

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## ENGLISH

**MD** Medical Device  
**LATEX** Caution: This device contains natural rubber latex which may cause allergic reactions.

### INTENDED USE

The device is intended to support drop-foot.

### Indications for use

Drop foot. Beware that the device must be reported to the manufacturer and relevant authorities. The patient should stop using the device and contact a healthcare professional:

- If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions.
- If any pain, skin irritation, or unusual reaction occurs with the use of the device.

### Warnings and Cautions:

- Never wear the device directly over an open wound.
- Do not take the device over the shoulder to over tighten the device.
- Patients with already compromised blood circulation in their extremities, like those with diabetes or peripheral vascular disease, need to be extra careful and are advised to consult with a healthcare professional before using the device.
- When you experience pain, swelling, sensation changes, or if your extremity shows signs of insufficient blood flow (turns blue, white, or cold) while using this device, discontinue the use immediately and consult with a healthcare professional.
- The device is not intended for use by patients with spasticity.

### GENERAL SAFETY INSTRUCTIONS

Read these instructions carefully before use. Keep them for future reference.

### USAGE

#### Cleaning and care

- Hand-wash using mild detergent and rinse thoroughly.

#### Note:

Do not machine-wash, tumble dry, iron, bleach, or wash with fabric softener. Note: Avoid contact with salt water or chlorinated water. In case of contact, rinse with fresh water and air dry.

### DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

### LIABILITY

Ossur does not assume liability for the following:

- Device not maintained as instructed by the user.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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