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polfit 17 polfit 22

polfit 19 polfit 21

REF.P1117 - Polfit 17 Wrist immobilizer with thumb, open; with rigid and malleable stays

REF.P1119 - REF.P1121- Polifit 19/21 Short/long size wrist immobilizer, open; with rigid and malleable stay

REF. P1122 Polfit22 Long-size wrist immobilizer with thumb support, open; with rigid and malleable stays

ORTHOservice RO+TEN Take care feel better

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EN – The descriptions and images in this document are for illustrative and commercial purposes only. Orthoservice reserves the right to make modifications without prior notice based on their needs.

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PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM

REFP1117 - Wrist immobilizer with thumb, open, with rigid and malleable stays
REFP1119 - REFP1121 - Short/long size wrist immobilizer, open, with rigid and malleable stays
REFP1122 - Long-size wrist immobilizer with thumb support, open, with rigid and malleable stays

Table with 5 columns: Code, Size, Wrist circ. cm, Length cm, Colour. Rows for REFP1117, REFP1119, REFP1121, REFP1122.

MAINTENANCE: Do not bleach, Handwash at max. 30°C, Do not use tumble-dry...
WASHING INSTRUCTIONS: Remove the splint. Hand wash in lukewarm water with mild soap (at max. 30°C); rinse thoroughly...

INDICATIONS: Wrist sprain, Wrist tendinitis, Arthritis and arthrosis, After wrist fractures or plaster's early removal...
CONTRAINDICATIONS POLFIT 17 / 19 / 21 / 22: None known

CHARACTERISTICS AND MATERIALS: Aluminium thumb splint, preformed and mouldable on medical prescription...
POLFIT 17/22: Balaie rigide en aluminium pour poaue, préformée et modélable, sur ordonnance...

PUTTING ON THE APPLIANCE POLFIT 17 + POLFIT 22: Adjust the palm metal splint to adapt it to the patient's wrist after removing it from the brace. Put the splint back into the brace (Fig. 1). Repeat the procedure for the thumb metal splint...
PUTTING ON THE APPLIANCE FOR THE PATIENT: Open all the Velcro straps and pull the straps out of the buckle (fastening them provisionally on themselves)...

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